

# RESPIRATORY PRODUCT



With N95 Replacement Filter



**BSN95R00**  
replacement N95 Mask

**Model:**  
**BSN95M00 OR BSN95MC0**  
42CFR84 Particulate Respirator  
**With VALVE**  
10 sets per box



**BSN95RC0**  
replacement N95 Mask with Carbon

## Special Features



Filter can be replaced to save cost




Valve respirator enhance wearers comfort



Mesh shell prevent mask collapse




Adjustable headband length for optimum fit, and hang over the neck

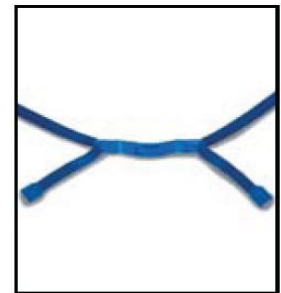


Quick Clip-Lock for easy donning


### Filter Replacement Instructions

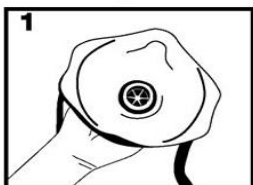

1 Hold the mask with inside upward

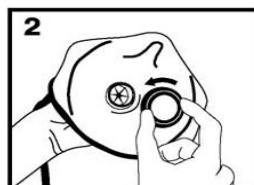

2 Unwind the valve lock to release the N95 filter

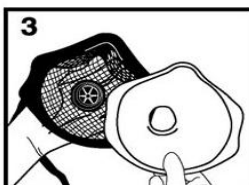

3 Remove the used N95 filter and replace a new one. Put the valve lock back and screw tight

NIOSH Approved Product



HQ: 123, Persiaran Pulau Pinang, Off Jalan Meru, 41050 Klang, Selangor, Malaysia.

Tel: 603-3342 3400 Fax:603-3342-3401

Penang: No. 6, Jalan Kelisa Emas 2, Tmn Kelisa Emas, 13700 Seberang Jaya, Penang, Malaysia.

Tel: 604-3987006/8 Fax:604-3987032